

Adjustable Folding Backrest




PRODUCT CODE: BACKREST

The adjustable folding backrest is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service. Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

Assembly Instruction

1. Carefully unpack and examine the product to ensure it is complete.
2. Before use check for any signs of damage e.g. cracked welding, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product.
3. Position Tube A as shown to desired angle, locate the Position Rail into one of the five height positions as shown.
4. The folding backrest can be adjusted from 1 to 5 positions by lifting the Position Rail and relocating it in any of the five height positions.

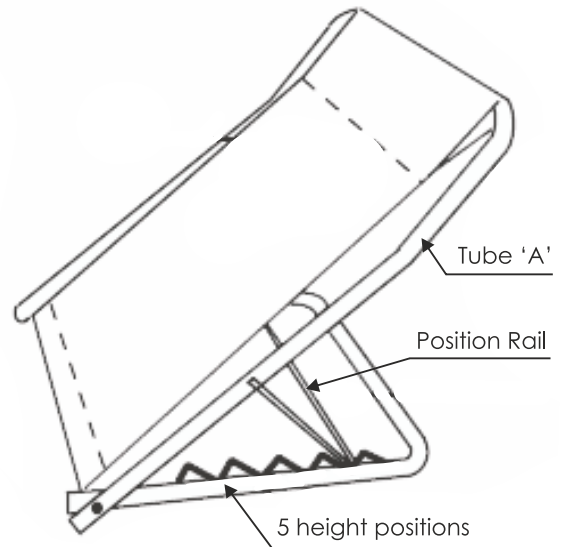

Correct Use

1. The adjustable folding backrest is designed to help the user obtain a comfortable position by means of partial support of body weight.
2. The backrest is not intended to support full body weight so do not lift full body weight via the product.
3. Do not attempt to overload the backrest.

Care & Maintenance

1. Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
2. Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.

The product should be checked on regular basis for damage or wear and tear. Any undue signs of damage should be noted and reported to the supplier in the first instance.

**N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the adjustable folding backrest for the particular user.
Weight Limit: 100 kg (16 st.)**

**N.B. Do not exceed the weight limit stated – doing so could put the user at risk.
N.B. Not suitable for use as a step, for use only by competent adults.**

**TOOL - FREE
ASSEMBLY**

Warranty/Guarantee Registration Card

Enter your details at www.kozeekomforts.co.uk/warranty

Petryl Enterprises Ltd t/a Kozee Komforts

Komfort House, Boundary Road, Bessingby Industrial Estate, Bridlington East Riding of Yorkshire YO16 4SD
Tel : 01262 409200 | Fax: 01262 602102 | E : sales@kozeekomforts.co.uk | W : www.kozeekomforts.co.uk

