

**Fitting and Maintenance Instructions**  
**ERTS-2 / ERTS-4 / ERTS-LID**

## INTRODUCTION

Thank you for deciding to purchase the Senator Raised Toilet Seat. The Senator Raised Toilet Seat is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

## INTENDED USE

Your Senator Raised Toilet Seat is constructed from recyclable ABS plastic; the joint is fused with a special ABS adhesive to form a watertight seal. The design of the seat enhances comfort and is simple to install and maintain. The securing fitments are colour coordinated to assist partially sighted users.

## WEIGHT LIMIT (WHEN PROPERLY INSTALLED)

The Senator Raised Toilet Seat can withstand a weight of 190 kg (30 stone). Please check that the toilet to which the Senator is to be fitted to is safe and secure. If in any doubt, contact a competent installer of sanitary fitments.

**NB. Do not exceed the weight limit stated – doing so could put the user at risk.**

## INSTALLATION

1. Take the blue coloured threaded bolt and install into the shaft of the 'L' shaped support bracket (See Fig. 1 and 2).
2. Place your Senator Raised Toilet Seat upside-down on a flat surface and install the brackets as shown (see Fig. 3) by rotating the hand wheel in a clockwise direction (see Fig. 4).
3. Raise or remove any existing standard toilet seat so as not to obscure the installation of your Senator seat.

**Fig. 1**




**Fig. 2**



**Fig. 3**



**Fig. 4**

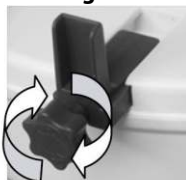

4. Place the Senator Raised Toilet Seat evenly onto the bowl of the toilet (see Fig.5).
5. Tighten both of the blue handles together (see Fig.6) making sure that the Senator seat remains evenly on the top surface of the toilet bowl (see Fig. 7).
6. Double check that the front skirt is flat against the bowl and check to see that it is secure (see Fig. 8).

**Fig. 5**



**Fig. 6**



**Fig. 7**



**Fig. 8**



## **CLEANING & MAINTENANCE**

Your Senator Raised Toilet Seat has been designed to give you many years of service, however it is essential that you look after your seat by following these simple instructions.

1. Clean your seat using a mild disinfectant or an antibacterial cleaning agent. Please check the suitability of the product before use.
  - a. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.
  - b. Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
2. The seat is suitable for industrial cleaning, but can be reissued only if the seal remains intact.
3. Dry your seat thoroughly before use.
4. We recommend that you regularly check your seat, ensuring that the hand wheels are tight and the seat remains evenly positioned on the bowl. If you suspect that your seat has moved out of position, please follow the instructions above to move your seat back into the correct position and retighten all fitsments.



**Under no circumstances must you stand on the seat or allow others to do so as this is highly dangerous and will compromise the safety of the user.**



**If you have to remove the Senator Raised Toilet Seat for any reason, please ensure that you follow the installation instructions again fully before using. Failure to do so could compromise the safety of the user.**

## **WARRANTY**

This product carries a 12 months warranty from the date of purchase against manufacturing defects. Misuse or Damage is not covered. This warranty does not affect your statutory rights.

## **MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Petryl Enterprises Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Petryl Enterprises Ltd, Komfort House, Boundary Road, Bessingby  
Industrial Estate, Bridlington YO16 4SD**

**Telephone: +44 (0) 1262 409 200 Fax: +44 (0) 1262 602 102**

**Web: [www.kozeekomforts.co.uk](http://www.kozeekomforts.co.uk) Email: [info@kozeekomforts.co.uk](mailto:info@kozeekomforts.co.uk)**