

RING CUSHION INSTRUCTIONS

1. Remove the ring cushion from the outer packaging.
2. Check that the cushion has a spare valve seat supplied (small black ring). Fig 1.
3. Unscrew the valve and check that there is a valve seat in the bottom of the valve.
4. Replace the valve plug, but DO NOT TIGHTEN
5. Gently separate the walls of the ring cushion, all the way round. This will make it easier to inflate. Fig. 2 & 3.
6. Inflate by mouth. Fig 4.
7. Tighten the valve plug.

Problem solving.

Fault	Cause	Remedy
Cushion will not inflate	Valve closed Cushion walls stuck together	Slacken valve and try again Separate walls and try again
Cushion deflates	Valve plug loose Valve seat missing/damaged Puncture	Tighten Replace valve seat with spare Return to supplier and purchase a new cushion/exchange.

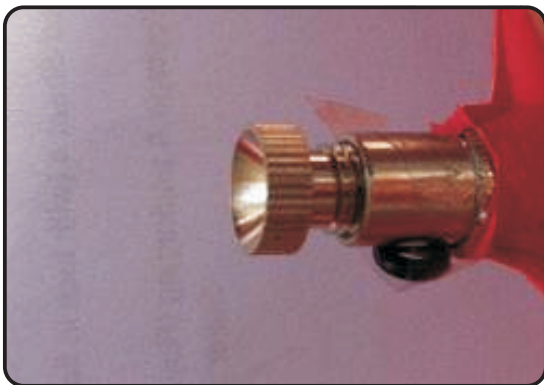


Fig. 1.



Fig. 2.



Fig. 3.



Fig. 4.