

Mud & Snow / Ice Grip

This snow & ice grip can be used with any walking stick. To attach to a walking stick, undo the screws, for maximum protection of your walking stick against scratching you may wish to place material / felt / foam (not supplied) around the stick where the Mud & Snow / Ice Grip attaches.

Attach the ice grip so the spiked base is flush with the bottom of the stick and, in effect, becomes the new base. Screw securely in place fastening the screws either side of the grip until completely tight, using the screwdriver.

The walking stick is now ready for outdoor use. When you return indoors, the two white levers can be squeezed together and the ice grip flipped up, positioning it along the stick, and thereby ensuring you don't damage any floors. You then simply use your stick as normal.

The plastic pads may be used to prevent damage to the stick or as spacers on narrow sticks.



Preparing to attach ice grip:
- screwdriver - ice grip - screws



Place body of ice grip under your walking stick making sure the base is flush with the ferrule.



Screw the two parts of the ice grip together.



Ice grip down for outdoor use.



Squeezing white levers together to lift ice grip up.



Ice grip up for indoor use.

Please note we do NOT recommend users of walking sticks to go out in the ice as this is the easiest time to slip and fall. However, should you wish or need to go out in wintry conditions, the ice grip may offer some additional stability in the ice. Please take extra care. Kozeekomforts cannot be held responsible for anyone who falls whilst using an ice grip.

Kozeekomforts

Komfort House, Boundary Road, Bessingby Industrial Estate,
Bridlington East Riding of Yorkshire YO164SD

Tel : 01262 409200 | Fax: 01262 602102
E : sales@kozeekomforts.co.uk
W : www.kozeekomforts.co.uk

