

Kozee 4 Wheel Rollator



PRODUCT CODE: ROLL176-RUBY | ROLL176-TITANIUM

The Kozee Rollator is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service. Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

Assembly Instruction

1. Remove all contents from the carton.
2. Open the rollator frame fully. Push the folding hinge until straight to secure the rollator in the correct position. Put the seat down.
3. Depress the pin clips on the backrest and slide it into the tubes at the front of the rollator. Ensure both sides of the backrest are in place with the pin clips protruding through the holes.
4. Insert a handle with the pre-assembled locking loop brake into each handle tube. Ensure the brake cable runs down the outside of the frame.
5. Adjust each handle to the desired height, lining up the holes appropriately. Slide the bolts through the holes from the inside of the frame and screw on the tightening knob until secure. Ensure the bolt heads are properly countersunk with the holes.
6. To adjust the position of the brake pad, undo the lock nut at the front of the brake handle and rotate the knurled nut, to the desired position. The back of the brake pad should clear the wheel by approximately 3mm when the brakes are not engaged. Tighten the lock nut.
7. Remove the protective plastic covering from the tyres.
8. Attach the basket to the frame. It can hang on the front of the rollator or under the seat.

Folding the Rollator

The basket must be removed before folding. The rollator is easily folded by lifting the seat and pulling the black strap attached to the hinge and the upped crossbar.

Hand Brake Operation

Note: Always engage and disengage brakes on both sides.

1. To apply the brakes, squeeze both brake handles. This can be used to slow and stop.
2. To lock the brakes, push both brake handles down until they snap into place. To release the locked brakes, squeeze both brake handles upward.

Adjust the Handle Height

Undo the tightening knob and remove the bolt. Adjust the handle to the desired height, lining up the holes appropriately. Replace the bolt and screw on the tightening knob until secure. Ensure the bolt heads are properly countersunk within the holes.

Sitting down

Before sitting, lock both brakes to prevent the rollator from moving. Place both feet firmly on the ground and do not lean backwards or sideways.

**N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the rollator for the particular user.
Weight Limit: 110 kg (17 st.)**

**N.B. Do not exceed the weight limit stated – doing so could put the user at risk.
N.B. Not suitable for use as a step, for use only by competent adults.**

**TOOL - FREE
ASSEMBLY**

Care & Maintenance

1. The rollator should be inspected before each use to ensure it is free from damage and excessive wear. Check that:
 - The handgrips are not loose or damaged, the brake cables are not damaged in any way.
 - The wheels are securely in place and are not damaged or excessively worn.
 - The frame is not cracked or dented, the frame is secure, stable, and all nuts and bolts are tight.**IF YOU DO FIND ANY WEAR OR DAMAGE TO YOUR ROLLATOR. DO NOT USE**
2. Check that all wheels and folding parts move freely.
3. Keep the walker clean by wiping with a damp cloth on a regular basis. (DO NOT USE abrasive cleaning materials).
4. Brake adjustment check (assembly instruction 6.).
5. If required, only use genuine parts supplied by Kozee Komforts or approved dealer.

Warranty/Guarantee Registration Card

Enter your details at www.kozeekomforts.co.uk/warranty

Petryl Enterprises Ltd t/a Kozee Komforts

Komfort House, Boundary Road, Bessingby Industrial Estate, Bridlington East Riding of Yorkshire YO16 4SD
Tel : 01262 409200 | Fax: 01262 602102 | E : sales@kozeekomforts.co.uk | W : www.kozeekomforts.co.uk

