Folding Shower Bench




PRODUCT CODE: SHOWERSTOOL-FOLD

The Folding Shower Bench is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service. Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

ASSEMBLY/HEIGHT ADJUSTMENT


- 1. Take all parts out of packaging and check before assembly.
- 2. Open the legs and locate the free leg into this aluminium keep.
- 3. Ensure that it is secure.
- 4. Identity both leas marked 1 and insert into the corresponding lea by depressing the sprung button.
- 5. Repeat for legs marked 2.
- 6. Ensure all buttons are fully located.
- 7. Check all legs are the same height.
- 8. Ensure the stool is stable and sturdy before use.

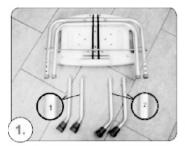







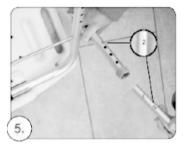







- N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the stool for the particular user.

 Weight Limit: 115 kg (18 st.)
- N.B. Do not exceed the weight limit stated doing so could put the user at risk.
- N.B. Not suitable for use as a step, for use only by competent adults.

TOOL - FREE ASSEMBLY

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair. Do not exceed 50C when cleaning the seat cover and/or back as this will permanently damage the plastic. Regular daily cleaning will reduce the build up of limescale on your product.

Warranty/Guarantee Registration Card

Enter your details at www.kozeekomforts.co.uk/warranty

Petryl Enterprises Ltd t/a Kozee Komforts

